Deutsch:

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9999-D0114-03 V007

English:

Instruction for use – Please read carefully

## IHDENT® Dentatrend® SURGOTOL Suction-System disinfection

### Type of preparation

High performance concentrate for the disinfection and cleaning of dental suction systems and the accompanying sinks.

### Application

The **SURGOTOL** Suction-System disinfection concentrate has to be diluted with water before usage. One can use concentrations between 1 and 3% of the final product (here 10 to 30 ml of concentrate is used to make 1 L of the final disinfection liquid). About 1 L of the final disinfection liquid is required for the correct disinfection of the suction-system. Depending on the bio load on the suction-system, disinfection is required 1-2 times a week. The concentrate may not be mixed with other cleaning agents or disinfection liquids / concentrates.

### Spectrum of activity

**IHDENT® Dentatrend® SURGOTOL Suction-System disinfection** acts against:  
Bactericidal according to VAH/DGHH, EN 13727, EN 14561, EN 1276. Levurocidal (Candida albicans) according to VAH/DGHH, EN 13624, EN 14562, EN 1650. "Limited virucidal" (active against enveloped viruses i.e. HBV, HIV, HCV) according to BGA(RKI)/DVV.

### Contact times

Bacteria and Candida albicans according to:  
VAH/DGHH (tested as instrument disinfection):

**high bio load** **2,0%** **5 min.**  
**1,0%** **60 min.**

Bactericide according to EN 1276:

**high bio load** **2,0%** **5 min.**

Candida albicans according to EN 1650

**high bio load** **2,0%** **5 min.**

Bactericide according to EN 13727, EN 14561

**high bio load** **3,0%** **2 min.**

Candida albicans according to EN 13624, EN 14562

**high bio load** **3,0%** **2 min.**

Enveloped viruses according to BGA(RKI)/DVV

**with bio load** **2,0%** **2 min.**

### Product composition

In 100 g SURGOTOL are:  
7,5 g dimethyldioctylammoniumchloride  
0,6 g benzyltrimethyldecylammoniumchloride

### Precautionary statements

- P264** Wash hands thoroughly after handling.  
**P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.  
**P332+P313** If skin irritation occurs: Get medical advice / attention.  
**P337+P313** If eye irritation persists: Get medical advice / attention.  
**P302+P352** IF ON SKIN: Wash with plenty of soap and water.

### Keep out of reach of children.

### Storage

The product can be stored at room temperature. The product may not be used once the expiry date has been reached.

### Disposal instructions

Packing material should be completely emptied of contents before disposal. Follow EWC 07 06 99 when disposing of any remaining product content.

### Special note

Always read the Instructions for Use and the Material Safety Data Sheet before use.

### Delivery units

2,5 L REF D0114

**Expert opinions:** Available on request.

### Legend

 Read instructions

 Expiration date

 Charge number

### Warning

**H315**  
Causes skin irritation.

**H319**  
Causes serious eye irritation.

CE 0029

9999-D0114-03

Revision issued 2015-07

IHDENT® and Dentatrend® are registered trademarks.

**Dr. Ihde Dental AG**  
Dorfplatz 11  
CH-8737 Gommiswald/SG  
Tel. +41 (0)55 293 23 23  
Fax +41 (0)55 293 23 00  
contact@implant.com  
www.implant.com

**Distribution by Dr. Ihde Dental GmbH**  
Erfurter Str. 19  
D-85386 Eching/München  
Tel. +49 (0)89 319 761-0  
Fax +49 (0)89 319 761-33  
info@ihde-dental.de  
www.ihde-dental.de

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