Deutsch:

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English:

Instructions for use – Please read carefully

IHDENT® E&B® 2

IHDENT® E&B® 2 is a simple to use self-etching dual cure adhesive. It is designed for strong bonding of composites, compomers and resin modified glass ionomer cements to enamel and dentin. IHDENT® E&B® 2 works optimally under slightly moist conditions, i.e. the dentin should not be dried thoroughly.

IHDENT® E&B® 2 consists of the components PART A and PART B that were mixed before application. It is bonding to dual cure, self-cure and light cure composite restorative materials.

Essential components

IHDENT® E&B® 2 contains modified polyacrylic acid, methacrylates, phosphoric acid esters, catalysts and stabilizers in ethanol.

Indication

IHDENT® E&B® 2 is used for light-curing and chemically curing (dual-curing) composites, compomers and resin-reinforced glass ionomer cements.

Contraindication

Do not use IHDENT® E&B® 2 if a patient is known to be allergic to any of the materials ingredients or if the stipulated working technique cannot be used. IHDENT® E&B® 2 is a light-curing adhesive only and should not be used in conjunction with self-curing resin materials. Dual-curing materials must be light initiated.

Side effects

IHDENT® E&B® 2 may cause a sensitizing reaction in patients with a hypersensitivity to any of its ingredients. In these cases, the material should no longer be used.

Interaction

Materials containing eugenol or clove oil may inhibit the polymerization of IHDENT® E&B® 2.

Application

1. Isolation

Rubber dam is the recommended method of isolation.

2. Cavity Preparation

Clean the tooth with flour of pumice and water prior to preparation. Prepare the cavity with minimal tooth reduction. Margins should have a slight (0.5 - 1.0 mm) bevel placed in the enamel to increase the surface area for greater bond strength.

3. Pulp Protection

Cavity floor of deep excavations should be covered with a thin layer of hard setting calciumhydroxide material.

4. Application of IHDENT® E&B® 2

One drop of IHDENT® E&B® 2 PART A and one drop of IHDENT® E&B® 2 PART B were combined in a mixing pallet and mixed for 5-10 seconds. Apply the homogeneous mixture generously with a brush onto the slightly wet enamel- and dentin surfaces for 30 seconds with agitation. The material should build a homogeneous layer. Air thin for 10 seconds to remove the volatile components and to disperse the adhesive. Then light cure for 20 seconds with a dental halogen light unit or an LED (wavelength 400–500 nm, light intensity min. 1000 mW/cm²) and place the restorative material. For a maximal adhesion it is strongly recommended to use the light cure mode. If light cure is absolutely impossible, the adhesive will also cure in the autocure mode. After application and inoculation of the adhesive, all volatile components must be removed through careful air application. A sticky layer must remain. Apply the restorative material directly on this layer.

Note: Do not interchange lids of the bottles, because this can lead to a cross-contamination of the liquids.

Best results are obtained with application of a thin layer of a flowable composite followed by the application of a moldable composite.

Light cure each composite layer separately according to the corresponding user instructions.

5. Restorative Placement, Cure and Finishing

Refer to manufacturer instructions for placement, curing and finishing of the used restorative materials.

Precautionary statements

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P262 Do not get in eyes, on skin, or on clothing.

Storage

Do not store above 25 °C (78 °F)! Avoid storage in direct sunlight. Do not use after expiration date.

**Keep away from children!
For dental use only!**

Pack IHDENT® E&B® 2

2 bottles, each 5 ml
16 Mixing pads, 1 Brush holder
50 Application tips

REF 5143

Legends

	Read instructions
	Expiration date
	LOT Charge number
	Store protected against the sun
	Keep in a dry place
	Store tightly closed keep
	Temperatures up to 25 °C
	Manufacturer

CE 0029

Danger

	H225 Highly flammable liquid and vapour.
	H315 Causes skin irritation.
	H317 May cause an allergic skin reaction.
	H319 Causes serious eye irritation.
	H335 May cause respiratory irritation.

9999-5143-03

Revision issued 2015-07

Dr. Ihde Dental AG
Dorfplatz 11
CH-8737 Gommiswald/SG
Tel. +41 (0)55 293 23 23
Fax +41 (0)55 293 23 00
contact@implant.com
www.implant.com

Distribution by

Dr. Ihde Dental GmbH
Erfurter Str. 19
D-85386 Eching/München
Tel. +49 (0)89 319 761-0
Fax +49 (0)89 319 761-33
info@ihde-dental.de
www.ihde-dental.de

IHDENT® and E&B® are registered trademarks.

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