Deutsch:

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English:

Instructions for use – Please read carefully

IHDENT® GIZ® Silver Powder & Capsules

IHDENT® GIZ® Silver is a chemically curing glass-ionomer cement for underfillings and constructions under crowns and bridges. IHDENT® GIZ® Silver is also used in restorative dentistry for fillings in deciduous teeth and for class I fillings.

Significant components

Ionomer glass, silver alloy and polyacrylic acid

Application

1. Capsules

After the preparation of the cavity or tooth stump, the surface to be treated is thoroughly cleansed and, if necessary, pretreated with GIZ® conditioner (REF 2330). Afterwards the surface is rinsed with water and well dried.

The capsule is inserted into the application forceps with the rear ridge and activated by squeezing vigorously once. Afterwards the capsule is immediately placed in the capsule mixer and mixed for approximately 10 seconds. After mixing, the capsule is again inserted in the application forceps, the front closure is removed from the protective sleeve and the capsule is emptied by squeezing numerous times. The cement is thereby directly applicated into the cavity.

2. Powder / Liquid

mixture ration:

4 Dosage spoon powder: 3 drops liquid (2.8 g powder: 1 g liquid)

The powder is admixed to the liquid in numerous small portions in order to improved the wettening of the powder. IHDENT® GIZ® Silver is introduced into the cavity with the Heidemann spatula and well adapted. Water (saliva) should be kept away from the filling until the end of the initial curing time since a low quality filling structure can result from a further absorption of water. In order to prevent the water from entering, it can be covered with bonding or fissure sealant or a layer of vaseline can be applied. Matrixes are used in the area of the approximal surfaces.

Processing

For use as stump material, the excess material can be removed with a high-grade corundum sander or flexible grinding wheels with vaseline. An impression can be performed immediately. When used as a filling material, polishing should be first performed after 8 minutes. Grinding stones or discs with vaseline should also be used here. Do not use sprays containing water. In order to protect IHDENT® GIZ® Silver from water in the first 24 hours, a varnish is applied to the surface.

Processing time

At a room temperature of 23 °C the following times (in minutes) are available for the individual phases:

- 00.00 Begin mixing
- 00.30 End of mixing (Capsule 00.10)
- 02.00 End of processing time
- 06.00 End of setting

Elevated temperatures shorten the setting time, lower temperatures increase the time. Exceeding the processing time leads to a significant loss of the adhesion to enamel and dentin.

Note

1. IHDENT® GIZ® Silver adheres very well to metal instruments and should be thoroughly removed prior to setting.
2. The powder must be tightly closed and stored in a cool and dry location.
3. IHDENT® GIZ® Silver can be used for two years after the date of production. After this period a reduction of the adhesion is to be expected and the liquid tends to become yellow after long storage periods.

Side effects / Interactions

Irritation of the pulpa can occur from all glass ionomers. A sensibilization due to IHDENT® GIZ® Silver cannot be ruled out in sensitive patients. Should such symptoms occur, the use is to be discontinued. IHDENT® GIZ® Silver may not be used if an allergy may be present. If IHDENT® GIZ® Silver must be removed in such cases, the removal may only be performed with a cofferdam in order to prevent an inoculation of components of this substance in the mucous membranes. Pulpa protection: IHDENT® GIZ® Silver can generally be applied directly to the dentin. If there is merely a thin layer of dentin before the vital pulpa, it is recommended to apply an underfilling containing calcium hydroxide.

Warning

IHDENT® GIZ® Silver contains a weak organic acid. Avoid the direct contact with the unprotected skin or the eyes as well as the mucous membranes with the unpolymerized cement or with the liquid. If contact does occur, immediately rinse thoroughly with water and consult a physician if necessary.

Precautionary statements

P280 Wear protective gloves / protective clothing / eye protection / face protection.

Store out of reach of children. Only for use by the dentist.

Storage

Do not use after the expiration date. The shelf life of an unopened package is 6 months as long as it is stored dry and at room temperature. See barcode on the package for charge and expiration date.

Package sizes

1. Capsules

Package with 50 capsules

1 capsule contains:

475 mg powder and 85 mg liquid **REF 9825**

2. Powder / Liquid

1 Package contains:

30 g IHDENT® GIZ® Silver powder

15 ml IHDENT® GIZ® Silver liquid **REF 9820**

Legend

-  Read instructions
-  Expiry date
-  Charge number
-  Keep in a dry place
-  Store tightly closed keep
-  Temperature up to 25°C
-  Manufacturer

CE 0029 ISO 9917-2:1998

Warning

H317

May cause an allergic skin reaction.

H319

Causes serious eye irritation.

9999-9825-03

Revision issued 2015-07

Dr. Ihde Dental AG

Dorfplatz 11
CH-8737 Gommiswald/SG
Tel. +41 (0)55 293 23 23
Fax +41 (0)55 293 23 00
contact@implant.com
www.implant.com

Distribution by

Dr. Ihde Dental GmbH

Erfurter Str. 19
D-85386 Eching/München
Tel. +49 (0)89 319 761-0
Fax +49 (0)89 319 761-33
info@ihde-dental.de
www.ihde-dental.de

IHDENT® and GIZ® are registered trademarks.

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