Deutsch:

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English:

Instructions for use – Please read carefully

IHDENT® Ionomolar®

Ionomolar® is an x-ray opaque, light curing single compomer filling material. It combines the advantages of glass ionomer cements - adhesion to tooth substance, fluorine release and biocompatibility - with those of light curing components, mechanical stability, aesthetic and color stability. After curing, the filling material can be highly polished.

It contains x-ray opaque glass filling materials in a matrix of polymerizable methacrylates.

Constituents

Ionomer glass in a matrix of dental resins

Area of application

- Restoration of deciduous teeth
- Restoration of class III cavities
- Restoration of class V cavities and cervical abrasions and erosions

Use

Color selection

Prior to the color selection and preparation, the tooth is to be cleansed of residues, concrements etc. Determine the color of the still moist tooth and choose a suitable Ionomolar® color. The compomer is colored with VITA colors.

Preparation

The preparation of the tooth and the cavity should be limited to a minimum. Wedge shaped preparations should be made in cervical cavities.

Cleansing

The cleansing of the tooth surface is very important for the adhesive formation. The cleansing of the tooth surface or cavity preparation is decisive for the adhesion and durability of the compomer filling.

In cases in which no cavity preparation is necessary (i.e. cavity-free cervical defect), clean the tooth surface with a rubber cup and pumice paste. Residues are to be thoroughly removed with water and dried with oil free pressurized air for approx. 10 seconds. It is thereby important to avoid drying out the dentin. To achieve better results, an enamel and dentin etching (15 seconds) is recommended. After etching, rinse with water for 15 seconds and afterwards dry with oil free pressureized air for approximately 10 seconds.

Contamination with saliva after preparation of the cavity or cleansing of the surfaces is to be avoided. Therefore a drain is to be performed with cotton rolls or cofferdam, depending on the clinical situation.

Protection of the pulpa

In deep cavities, especially with a remaining thickness of less than 1 mm, the areas near the pulpa are to be covered with a curing calcium hydroxide underfilling. As much dentin surface as possible should remain free for the bonding with Ionomolar®. Direct and indirect pulpa cappings with Ionomolar® are not indicated.

Bonding

The use of IHDENT® P&B® (REF 5120) is recommended. This single component bonding system is distributed over the entire cavity surface with a brush or sponge and blown out to a thin layer. Afterwards, IHDENT® P&B® is cured with light for 20 - 40 seconds.

Curing

For the polymerization of Ionomolar®, layer thicknesses of not more than 2 - 3 mm should be chosen. Every layer is cured with light for 40 seconds. It is important to thoroughly cure all areas of the restauration. Dark colors of Ionomolar® require longer curing times of at least 60 seconds.

Finishing / Polish

The finishing of the filling is performed with polishing discs and / or rubber polishers. A high shine can be achieved with soft shine polishers or with oral polishing felt.

Side effects

A sensitization in sensitive patients due to compomeres cannot be ruled out. Should corresponding reactions occur, the use is to be discontinued. In a proven allergy, fillings of Ionomolar® may only be removed with a cofferdam protection in order to avoid encapsulations of particles from filling components in the mucous membrane. A prophylactic removal without cofferdam is therefore strictly contraindicated.

Contraindications

Direct or indirect pulpa cappings.

Warning

- Do not mix the compomer with resins or other compomer materials in order to adjust the viscosity of the compomer material.
- Materials containing eugenol may not be used with Ionomolar®, since these impair the curing and can lead to softening of the filling or to color alterations of the compomer. Also not sealed products containing eugenol in the vicinity can be damaging; therefore do not allow uncured Ionomolar® to contact products containing eugenol.
- The product is not suitable for direct or indirect pulpa cappings as well as for chewing force loaded fillings of the remaining dentition.

- Avoid contact with the eyes and skin, especially in persons who have known allergic reactions to methacrylate. Upon contact with the eyes, rinse thoroughly with water and consult a doctor. Upon skin contact, immediately wash thoroughly with water and soap. Irritates the pulmonary organs.

Storage

See package or barcode for charge number and expiration date. After opening of the secondary package (aluminum bag), the material can be used for a maximum of 6 months. The maximum storage temperature is 25 °C. Protect from moisture.

Note

Keep out of reach of children. Only to be used by the dentist.

Package sizes

Syringes (yellow) with 3.5 g contents	REF
universal A2	2821
brown A3,5	2825
universal A3	2826
20 Tips à 0,3 g contents	
universal A2	2828
brown A3,5	2832
universal A3	2833

Legends

	Read instructions
	Expiration date
	LOT Charge number
	Keep in a dry place
	Store tightly closed keep
	Temperatures up to 25 °C
	Manufacturer

CE 0029

ISO 9917-2

9999-2821-03

Revision issued 2015-08

Dr. Ihde Dental AG
Dorfplatz 11
CH-8737 Gommiswald/SG
Tel. +41 (0)55 293 23 23
Fax +41 (0)55 293 23 00
contact@implant.com
www.implant.com

Distribution by

Dr. Ihde Dental GmbH
Erfurter Str. 19
D-85386 Eching/München
Tel. +49 (0)89 319 761-0
Fax +49 (0)89 319 761-33
info@ihde-dental.de
www.ihde-dental.de

IHDENT®, Ionomolar® and P&B® are registered trademarks.

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Vita® is a registered trademark of the Vita Zahnfabrik H. Rauter GmbH.