Deutsch:

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English:

Instructions for use – Please read carefully

IHIDENT®

JetBond II

JetBond II is an automix non-eugenol temporary crown and bridge cement suitable for cementing temporary crowns or bridges as well as for trial cementing of permanent restorations. It shows excellent adhesion to temporary crown and bridge materials, but it can easily be removed from prepared tooth stumps. JetBond II is delivered in AUTOMIX-cartridges so it is very easy and comfortable to use. Due to its non-eugenol formulation it does not influence the setting of resin-based filling and luting materials. The temporary crown and bridge cement is based on ZnO/organic acids. Especially in cases of allergies to eugenol it is strongly recommended to use this eugenol-free type of cement.

Application

Application gun and cartridge:

Load the application gun with 1:1-plunger and remove and dispose the closure-cap. Do not re-use the cap! Install the mixing cannula onto the cartridge and extrude needed material. After each use leave the used mixing cannula in place to seal the cartridge.

Note:

The initial extrusion from the mixing cannula (about the size of a pea) should be discarded. Then the following mix will be perfect. This must be done for each new mix.

Cementing

JetBond II can be applied directly into the temporary crown or bridge under slight pressure from the mixing cannula.

Use a thin layer to the inside of the temporary crown or bridge and seat not later than 1:30 minutes (working time) cautiously but firmly over the prepared tooth structures.

Note:

Do not dry stumps, best moisten them with some water. A humid milieu (humid stumps) additionally facilitates to remove the temporary crown or bridge later on.

JetBond II sets in the mouth within 2-3 minutes. After JetBond II has set remove any excess.

Warning

- Do not use JetBond II if in some cases individual contact allergies may exist.
- In case of contact with skin and eyes rinse immediately with plenty of water and consult a physician if necessary.
- If swallowed seek medical advice.

Precautionary statements

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Keep away from children!

For dental use only!

Storage / Shelf life

Store at 8 - 25 °C (47 - 78 °F). Store at dry place; store away from food stuffs. Do not use after expiry date.

Disposal considerations

Use proper landfill disposal or incineration in accordance with local, state and federal regulations.

Package sizes

2 cartridges à 25 ml and 12 mixing cannulas, blue

REF 2126

Legends

	Read instructions
	Expiration date
	LOT Charge number
	Keep in a dry place
	Store tightly closed keep
	Temperatures up to 25 °C
	Manufacturer

CE 0029

Warning

H315

Causes skin irritation.

H319

Causes serious eye irritation.

H335

May cause respiratory irritation.

H400

Very toxic to aquatic life.

H410

Very toxic to aquatic life with long lasting effects.

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IHIDENT® is a registered trademark.
Dr. Ihde Dental AG