Deutsch:

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English:

Instructions for use – Please read carefully

# IHDENT® Mastertray®

Mastertray® is a light-cure impression tray material. The resin is slightly transparent, blue or pink. The thickness of the light-cure tray material is 2 mm. Advantages of Mastertray® are less shrinkage, resistance to water absorption and high dimensional stability.

## The most important constituents

Megatray® consists of a matrix of polyfunctional methacrylates and inorganic filler.

## Indication

Preparation of custom trays, bite plates and base plates for needle-point tracing.

## Application

The plaster model must be insulated with alginate insulation before using Mastertray®. The cover-fleece from the wafer has to be removed, than Mastertray® is moulded onto the model. The tray can be cut to the desired rim. Use excess material to form the handle.

The wafer is polymerized in a UVA or blue-light unit for 5 minutes. The wafer is taken off the model and the reverse side of the tray is cured for another 5 minutes. After polymerising, the inhibition layer caused by the airoxygen is removed by a tissue moistened with an alcoholic cleaner easily (e.g. IHDENT® Dentatrend® Disinfection wipes, REF D0032). Use a dust extractor for trimming with carbide burrs.

## Disinfection

The custom trays of light-cure impression tray material can be disinfected with any commercial standard Impression disinfection (e.g. IHDENT® Dentatrend® Impression Disinfection, REF D0210). The duration depends on the data of the respective manufacturer.

## Warning notes

Mastertray® contains monomers for polymerization (e.g. methacrylic esters) which can cause skin allergies, especially with sensitive patients. Do not use the product where allergic reactions are observed, or for patients with methacrylate/acrylate-based resin allergy history.

**Keep out of the reach of children. To be used by dentists or dental laboratories only.**

## Storage

Store protected against the light and not above 25 °C (77 °F). Batch number and expiry date see label or bar code on the package.

## Presentation

Package with 50 wafers  
upper jaw, blue  
upper jaw, slightly transparent  
upper jaw, pink

REF 7005  
REF 7010  
REF 7015

## Legends

	Read instructions
	Expiration date
	LOT Charge number
	Store protected against the sun
	Keep in a dry place
	Store tightly closed keep
	Temperatures up to 25 °C
	Manufacturer

9999-7010-03

Revision issued 2015-08

## Dr. Ihde Dental AG

Dorfplatz 11  
CH-8737 Gommiswald/SG  
Tel. +41 (0)55 293 23 23  
Fax +41 (0)55 293 23 00  
contact@implant.com  
www.implant.com

## Distribution by

## Dr. Ihde Dental GmbH

Erfurter Str. 19  
D-85386 Eching/München  
Tel. +49 (0)89 319 761-0  
Fax +49 (0)89 319 761-33  
info@ihde-dental.de  
www.ihde-dental.de

IHDENT®, Mastertray® and Dentatrend® are registered trademarks.

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