Deutsch:

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English:

Instructions for use – Please read carefully

IHDENT®

Relation D

Relation D is an elastomere bite registration material with a silicon base, addition curing, with extremely high Shore hardness (only measurable according to Shore D).

The special feature in Relation D is its optical reflection activity; it is particularly suitable for powder-free optical 3D registration and data acquisition in CAD/CAM/CIM areas of application.

Constituents

Polyvinyl siloxane

Characteristics of the material

The material is characterized by its good millability due to the very high final hardness.

Simple application is possible due to the thixotropy. The good stability prevents flowing into the interdental spaces and allows a simple removal from the mouth. The produced bite registers are permanently dimension stable.

Therapeutic indications

Injectable material for the bite registration; also usable in closed bite position.

Use

1. Insert cartridge in the mixing gun.
2. Remove cartridge seal and dispose of. Do **not** reuse the seal.
3. Press out Relation D until (however without the mixing cannula), material flows equally from both chamber.
4. Attach mixing cannula to cartridge and press out required amount of Relation D. **Note:** Only use mixing cannulas of the same color as the seal of the cartridge or turquoise colored mixing cannulas!
5. Leave filled mixing cannula on the cartridge after use.
6. Prior to the next use remove mixing cannula and check the discharge opening of the cartridge for blockage.
7. Attach new mixing cannula and proceed as described.

Processing time: at least 30 seconds

Retention time in mouth: at least 30 seconds

Total setting time: at least 60 seconds

All information based on 23 °C (73.4 °F), approx. 50 % relative humidity.

Physical characteristics

Hardness according to Shore D: approx. 45
linear size alteration: approx. 0.1 %

Disinfection

Bite registers can for example be disinfected in 2 % acidic glutardialdehyde solution.

Side effects

None known

Contraindications

None known

Interactions

None known

Warning

Solutions used prior to the registration (i.e. mouth washes) can disturb the setting reaction of the registration material and must be removed with thorough spraying and drying.

Do not leave any residues from the registration material in the oral cavity. Avoid contact with clothing since the material can not be removed through chemical cleaning.

Keep out of reach of children. Only to be used by the dentist.

Storage / Shelf life

Store at 5 - 27 °C. See label or barcode on the package for shelf life and expiration date. Do not use after expiration date.

Package sizes

2 cartridges à 50 ml and 12 mixing cannulas, turquoise **REF 7780**

Accessories

Safety mixing gun (1:1)
24 mixing cannulas, turquoise

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IHDENT® is a registered trademark.
Dr. Ihde Dental AG
Dr. Ihde Dental GmbH