Deutsch:

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English:

Instructions for use – Please read carefully

# IHDENT® TAK®

TAK® is a flowable, light-curing and radiopaque one-component compomer filling material. TAK® combines the advantages of glass ionomer cements (adhesion to the tooth structure, fluoride release and biocompatibility) with those of light-curing composite materials (handling during application, mechanical stability, aesthetics and shade stability). TAK® can be polished to a high lustre after curing.

## Essential components

Contains glass fillers in a matrix of polymerisable monomers based on acrylic resin.

## Range of indications

- Restoration of deciduous teeth
- Class III anterior fillings
- Restoration of Class V cavities (cervical caries, root erosion, wedge-shaped defects)
- Temporary fillings.

## Application

### Preparing the tooth

Before taking the shade and preparation, clean any debris and calculus etc. from the tooth. Then determine the shade of the tooth while it is still moist and select the appropriate TAK® shade (Vita® shades). TAK® is used after the area has been etched, washed, dried and a prior application of bonding agent using IHDENT® P&B® (REF 5120). Apply TAK® to the dentine and enamel in the selected shade. The material can either be applied directly from the syringe using the application tip supplied or a plastic spatula. Transparent strips can be used for contouring. Placement of a cavity liner is not required if there is a residual thickness of 1 mm. It is essential to avoid contamination with saliva after preparing the cavity and cleaning the surface. The site should therefore be isolated with cotton wool rolls or a rubber dam, depending on the clinical situation. In the case of deep cavities, cover areas near the pulp with a self-curing calcium hydroxide cavity liner. Direct and indirect pulp capping is contraindicated with TAK®.

### Curing

We recommend light curing lighter shades (e.g. A1; B1) for 20 - 40 seconds and darker shades (e.g. A3.5; B3) for 40 - 60 seconds.

Note: If the restoration is thicker than 2.0 mm, we recommend curing in increments to minimise polymerisation shrinkage.

### Finishing

Prepare with fine-grit diamond instruments, Arkansas stones or abrasive discs. Polish to a high lustre using soft polishers or oral felt polishers. Metal strips (e.g. Dremel No. 1 and No. 2) should be used interdentally.

### Side effects

Irritation is possible on direct contact with the pulp. The floor of deep cavities should, therefore, always be covered with a thin layer of self-curing calcium hydroxide cavity liner to protect the pulp. It cannot be ruled out that TAK® may cause sensitisation in sensitive patients. Should this occur discontinue using TAK®. If there is an established allergic reaction, a rubber dam should be placed as protection before removal of TAK® fillings to prevent components of the material infiltrating the oral mucosa. Prophylactic removal of TAK® without the protection of a rubber dam is therefore absolutely contraindicated.

### Contraindications

The product is unsuitable for direct or indirect pulp capping as well as occlusally load-bearing fillings in the permanent dentition.

Avoid contact with the skin, particularly with patients who have a known allergy to methacrylate.

### Interactions

The consistency of the compomer material must not be modified with bonding agent or diluted with resin in order to adjust the viscosity of the material. Materials that contain eugenol must not be used together with TAK®, as they could impair curing and soften the filling

or alter the shade of the compomer. Proximity to open containers of products that contain eugenol could also be detrimental; consequently, avoid contact of uncured TAK® with products that contain eugenol.

**Keep out of reach of children.  
Intended only for use by dentists.**

## Storage / Shelf life

Store at 23 °C. Cool storage extends the shelf life. Protect from moisture. Refer to the barcode on the packaging and syringe label for Batch Number and Expiry Date.

## Pack size

5 g syringe	REF 2811
Shade A2	REF 2815
Shade A3.5	REF 2816
Shade A3	REF 2816

3 Application tips

### Legend

	Read instructions
	Expiration date
	LOT Charge number
	Keep in a dry place
	Store tightly closed keep
	Temperatures up to 23 °C
	Manufacturer

CE 0029

## DIN EN ISO 4049

9999-2811-03

Revision issued 2015-08

**Dr. Ihde Dental AG**  
Dorfplatz 11  
CH-8737 Gommiswald/SG  
Tel. +41 (0)55 293 23 23  
Fax +41 (0)55 293 23 00  
contact@implant.com  
www.implant.com

## Distribution by

**Dr. Ihde Dental GmbH**  
Erfurter Str. 19  
D-85386 Eching/München  
Tel. +49 (0)89 319 761-0  
Fax +49 (0)89 319 761-33  
info@ihde-dental.de  
www.ihde-dental.de

IHDENT®, TAK® and P&B® are registered trademarks.

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Vita® is a registered trademark of Vita-Zahnfabrik.