Deutsch:

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English:

Instructions for use – Please read carefully

IHDENT® TempBase®

IHDENT® TempBase® is an automatically mixed two component material in cartridges based on multifunctional methacrylic esters. IHDENT® TempBase® is a very easy and comfortable material for creating short- and long-term temporary crowns or bridges, inlays, onlays and veneers. Because of its flexibility, the material is especially suitable for longer bridge spans. IHDENT® TempBase® shows fluorescence in UV-light and is free of methyl methacrylate. Its temperature derived while curing is lower than 40 °C. As a temporary crown or bridge it protects the prepared teeth against external influences and preserves the occlusion. It shows increasing transverse strength, good abrasion resistance and low polymerization shrinkage. It fits perfect. Showing good polishability, good color stability and fluorescence it looks perfect. And if IHDENT® TempBase® breaks - which may happen with acrylics - it is easy to repair. Cured material can be repaired with any natural or light cure composite building up again a chemical link. In nearly every property IHDENT® TempBase® is superior to the common materials based on monomer / polymer systems. It allows thin edges and production of longer bridge spans.

Impression taking

Before preparing the teeth or before a planned extraction of a tooth an impression is made by addition curing silicones or by alginate. Interdental areas are carved out.

Note: In the silicones impression block out undercuts and if necessary cut groves into the impression.

Preparing of the cartridge

The turn cap of the cartridge is taken off and thrown away (do not use it again!). It is substituted by the supplied mixing cannula by attaching it with a sideways turning movement, locking the cannula in place. The application gun is loaded with the prepared cartridge and is ready for application.

Note: The initial extrusion from the mixing cannula (about the size of a pea) should be discarded. Then the following mix will be perfect. This must be done for each new mix. Leave the used mixing cannula on the cartridge. It serves as a cap.

Application

IHDENT® TempBase® is automatically mixed when dispensed with slight and even pressure directly into the situation impression made before. Filling should occur from bottom upward to prevent voids.

Forming of the temporary crowns or bridges

- Load the situation impression with IHDENT® TempBase®.
- Seat the impression onto the prepared areas of the teeth. Step **a** and **b** must be done within 50 seconds (working time at 23 °C).
- After 1-2 minutes (setting time in mouth) the material shows a hardened but still elastic condition and can be removed from the teeth. (alternatively: 3-4 minutes setting time on the model)

Note: The temporary crown and bridge substitute can only be removed without destruction during the elastic state.

Post curing and finishing

After removal of the temporary crown or bridge from the impression (or from the tooth preparation) excess material and proximal undercuts are removed. Preferably, the temporary is then post cured in warm (45 °C - 55 °C) water (e.g. in a hot cure polymerization device). If this is not possible or not desired, after 6 minutes from beginning of application the temporary acrylic can be worked out with rotary instruments and can be polished to high gloss.

Do not breathe polishing dust; use suitable mouth protective device or aspiration!

Note: The oxygen inhibited smear layer on the surface caused by the acrylic system can not be fully avoided and should be removed before working out. It can easily be removed by alcohol or other suitable solvents.

Cementing of the temporary

IHDENT® TempBase® crowns or bridges preferably should be cemented with an eugenol-free temporary cement (e.g. Allfit Prov - REF 2135). In case of using eugenol-containing cements it should be considered that later used acrylate based cements could be hindered in hardening (inhibition of curing reaction by traces of remained eugenol).

Repairs

IHDENT® TempBase® show high mechanical strength.

However, if a temporary breaks the following procedures are recommended:

- Fracture shortly after production: Both ends of the fracture are refixed with freshly extruded IHDENT® TempBase®.
- Fracture of longer existing temporary: The areas of fracture are cleaned and roughened and provided with some mechanical retentions. Thus prepared, the fractured areas can be joint with freshly extruded IHDENT® TempBase®.

To enhance the curing place for some minutes in warm water.

Special hints

Non hardened IHDENT® TempBase® can easily be removed with alcohol or other suitable solvents.

Precautions

IHDENT® TempBase® is free of methyl methacrylate but contains other methacrylates. With susceptible patients, sensitization to IHDENT® TempBase® cannot be excluded. IHDENT® TempBase® should not be used any more, if allergic reactions are observed. Do not use for patients with allergic reactions against acrylates. Avoid contact with skin, mucous membrane and eyes. If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.

Precautionary statements

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P302+P352 IF ON SKIN: Wash with plenty of soap and water.

Storage

Store at room temperature (15-25 °C / 59-77 °F)! IHDENT® TempBase® shall not be used after expiry date!

Keep away from children!

For dental use only!

Package size

2 tubes (106 g base material, 6 g catalyst) and accessories
 shade universal (A2) REF 7040
 catalyst severally REF 7042

Legends

-  Read instructions
-  Expiration date
-  LOT Charge number
-  Keep in a dry place
-  Store tightly closed keep
-  Temperature range from 15 °C to 25 °C
-  Manufacturer

CE 0029

Warning

- H315**
Causes skin irritation.
- H317**
May cause an allergic skin reaction.
- H319**
Causes serious eye irritation.
- H335**
May cause respiratory irritation.

9999-7040-03

Revision issued 2015-08

Dr. Ihde Dental AG
 Dorfplatz 11
 CH-8737 Gommiswald/SG
 Tel. +41 (0)55 293 23 23
 Fax +41 (0)55 293 23 00
 contact@implantf.com
 www.implantf.com

Distribution by

Dr. Ihde Dental GmbH
 Erfurter Str. 19
 D-85386 Eching/München
 Tel. +49 (0)89 319 761-0
 Fax +49 (0)89 319 761-33
 info@ihde-dental.de
 www.ihde-dental.de

IHDENT® and TempBase® are registered trademarks.

Time Table of application and curing of IHDENT® TempBase®

temperature of the material [°C]	ca. 23 °C	ca. 37 °C	ca. 50 °C	ca. 23 °C
	0:00	ca. 0:50 min	ca.3 min	ca. 4:30 min
	application in mouth	setting in mouth (1-2 minutes)	completion of setting	trimming/ polishing

If the completion of setting takes place at room temperature, trimming / polishing could be done not before after 6 minutes (from start of mixing).

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